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HONSUN (Nantong) Co., Ltd.
No.8, Tongxing Road, Economic & Technical
Development Area, Nantong City, Jiangsu, P. R. China

Mfg. Date:

Please read the instructions thoroughly and carefully before using this product.

Application/safety: The diagnostic light may only be used by authorized specialist personnel. The instrument must be kept out of the reach of children.

Purpose: The light is used as an aid for diagnostic illumination of the area to be examined, such as the oral cavity or the skin surface.

Operation: Press the push-button at the upper end to switch the device on and off.

Inserting the batteries: Unscrew the black, upper end of the diagnostic light. Now insert two AAA batteries with the positive pole towards the bulb. You can then screw the upper end back in.

Warnings: Do not place the diagnostic light in liquids. Avoid direct contact with the tongue and mucous membrane during use. When examining the oral cavity, special care must be taken to ensure that the light does not fall into the oral cavity under any circumstances. There is a risk of suffocation! The trader accepts no liability for translation faults.

Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße 80, 20537 Hamburg, Germany
SRN: DE-AR-000000001
Art. No.: 5810035 Manufacturer No.: HS-401F2

MDR (EU)
2017/745
ISO 13485:2016;

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Fleischhacker GmbH & Co. KG